

Art.No 31-30-00

- Length wise foldable stretcher with upper body lifting. The frame is made of aluminium.
- Extendable handles in both ends.
- 2 wheels of PVC (Ø 100 mm) and 2 support feet of zinc coated steel.
- Patient restraints with plastic quick-release buckles.
- **Fabric:** Coated, flame retardant polyester fabric (meets DIN 5363 and DIN 4108 B1 standards).
- The stretcher meets standard EN 1865.

Users instructions

Grip in the middle of the frame tube and open the stretcher lengthwise straight. Check that the locks are engaged. Pull out the handles from the frames. Carry the stretcher always with patient's head in the direction of motion. Remember the safety of the patient!

Service instructions

After use clean the stretcher and check the mechanical functions. Check the condition and clean the stretcher every month, depending on the use.

Use of a damaged product is strictly forbidden.

Dimensions	Length (mm)	Width (mm)	Height (mm)
Opened	1900	550	130
Folded	950	550	210
Weight: 10 kg			
Carrying capacity: 150 kg			



Accessories

32-30-00	Patient belts 3 pcs + storage bag
00-31-41	Storage bag for Stretcher

Storage

The stretcher should be stored in a dry and clean place. For storage the stretcher can be folded.